www.medpip.com

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Some words with nurses, doctors, technicians and collegians

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According to recent studies, about 6% of people got wounded while opening an ampoule and this number is 26% for aged people.

Majority of laboratory and curative works specially the most common way of taking medicine which is injection are somehow dealing with sharp wastes.

Therefore, if the standards do not observed correctly there might be high risk of many diseases such as: Aids, Hepatitis B, Hepatitis C and also more than 20 other dangerous diseases for Doctors, Nurses, patients and even garbage handlers. In larger scale the whole society might be at risk.

POLE IDEAL PARS co. the winner of "Leading company in increasing of medical equipments quality" award from Medical Equipment Department of Health Ministry, started wide researches about ampoule opener device due to its goal to increase "National Health". Eventually this company introduces its Ampoule Opener product that is fully accorded to international standards of medical society.

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A real photo of a cut injury whilst opening a anaesthetiz ampoule by a specialist.
Aceration measuring: 20 mm, 5 mm, 5 mm

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1. Identify the breaking point of ampoule for breaking

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2. The head of ampoule should place correctly into the ampoule opener.

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3. The complex should keep according to the following picture.

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4. In order to break, push top of the ampoule with shown angle.

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5. Ampoule opener will separate from the body of the ampoule while the top is still in the ampoule opener.

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6. Separated part, should thrown in safety box.