

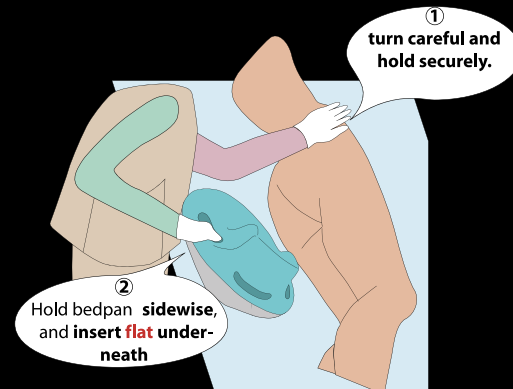


Simple to handle.

For lateral application the patient can be turned from or towards the nurse. Multiclean Bedpan can be used dorsal as well. Positioning the patient right on Multiclean Bedpan is done, when the human sacrum is lying right above the sacrum painted on Multiclean Bedpan.

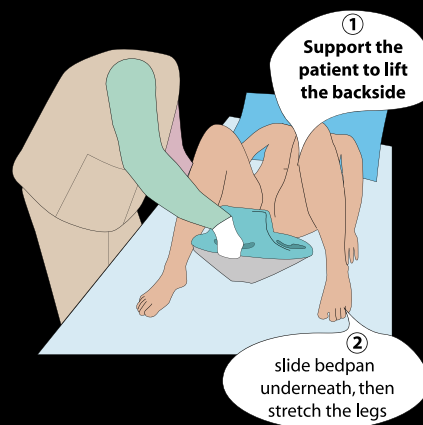

or

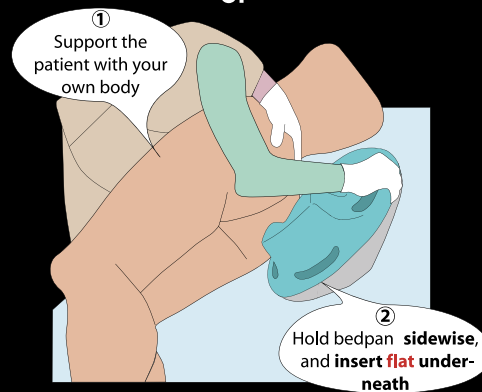

Pole Ideal Pars

www.medpip.com

PIP. Multiclean Bedpan



BRS BRS
ISO 9001 : 2008 ISO 13485:2003
FOR SAFETY BOX
www.medpip.com / info@medpip.com

1st floor , No.12, Naghdi St., Jahantab St.,
Mottahari Ave. Tehran, Iran
Tel: +98 21 88545922-9
Fax: +98 21 88765561 & +98 21 88767159
Postal code: 1576635714 P.O. Box: 15875-9483

● More comfort for your personnel and patients ● Cost-reduction for your institution

P.I.P. Multiclean Bedpan

Bedpan is a common way of gathering urinary and fecal discharge of bedridden patients and has no alternative. This wide range of patients including after-surgery convalescents, those who have movement disorders, generally all those patients who cannot use bathrooms by themselves, should use bedpan, so this has become one of the most common clinical operations. The design of current bedpans traces back to early 20th century which means more than 100 years ago and because of being outdated and inefficient, there has always been difficulties whether for patients or hospital personnel to use them, e.g. being inconvenient and causing pressure and pain in patients body, difficult for females to use, the aperture being too wide, being nasty and dirty (which makes it stressful for both patient and nurse), hard to discharge, etc.

Such problems are common and inevitable in clinical environments which have negative psychological effects on patients and nurses; therefore, with the purpose of providing patients and clinical personnel with more convenience, it was required to make bedpan application easier, more simple, and hygienic. In this regard, P.I.P. has provided a new definition of using bedpan through the accurate and ergonomic design of P.I.P. Multiclean Bedpan.

patients:
As comfortable as the toilet at home.
Useable regardless the sex

generally known- if someone passes faeces mostly urine is also passed. Thus men are left juggling with a urine bottle while lying on the bedpan.

P.I.P. Multiclean Bedpan - one piece for both and all, simultaneously

Protection of the intimate sphere. Reduction of unpleasant odours.

Conventional bedpans offer neither a visual cover nor protection from unpleasant odours. The patient is lying bare and naked above the bedpan. The ergonomic design and the splash protection cover of P.I.P. Multiclean Bedpan reduces unpleasant odours and hides from view as the only available system.

Secure application and control

Patients often clench while lying on a cold, hard steel bedpan. Being scared of something going aside only adds to this misery. Application is unfirm and patients can not stabilize themselves enough. The all-round sealing design and the splash protection cover of Multiclean Bedpan prevents of something going aside. Two side grips and the flat design of Multiclean Bedpan enable the patient to completely control Multiclean Bedpan.


P.I.P. Multiclean Bedpan: Ergonomic design High lying comfort minimizes the risk of pressure ulcer.

Lying on a steel bedpan is cold, hard and uncomfortable. The much too small edge (red) hurts after a little while and can cause pressure ulcer (decubitus ulcer). The patients weight is supported by a much too small surface, this is the reason for high point load and can cause pressure points on the patients backside. An often seen phenomena is that urine flows along the patients backside, this is a great danger for pressure ulcer. Multiclean Bedpan offers highest lying comfort by having a 3.5 times bigger surface. Multiclean Bedpan uses large, ergonomically formed areas which guarantee optimal weight distribution. Reduction of the point load is effective decubitus prophylaxis. The new vertebral column relieving immersion offers safety and absence of pain even after surgeries.



9. Removing without soiled beds - no «suction pad-effect»

Removing the bedpan can also soil the bed if it had adhered to the patients buttocks and detaches suddenly. Even if Multiclean Bedpan is tilted sideways during removal, its hollow body with two chambers makes sure, that nothing is spilled.


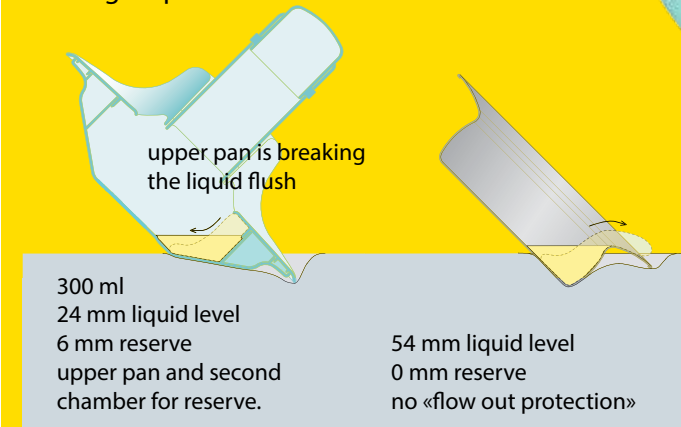


Essential reduced soiling of the buttocks. No super infections.

The buttocks immerse very deep into a conventional bedpan. The intaking hole is unnecessary wide. Patients are lying in their excrements and get soiled by their own excrements, this can cause a super infection. The opening of Multiclean Bedpan was designed to be as large as necessary. An immersion of the buttocks is impossible, the patient is lying above the excrements. This results less soiling and prevents super infections.

No soiled beds Controlled direction of expulsions

During use of a bedpan stool and urine, especially in case of diarrhoea or enema, can spill out of the bedpan and soil the bed linen. The splash protection cover of Multiclean Bedpan funnel the excrements into Multiclean Bedpan. This prevents the bed from getting soiled. The patient can press without danger.

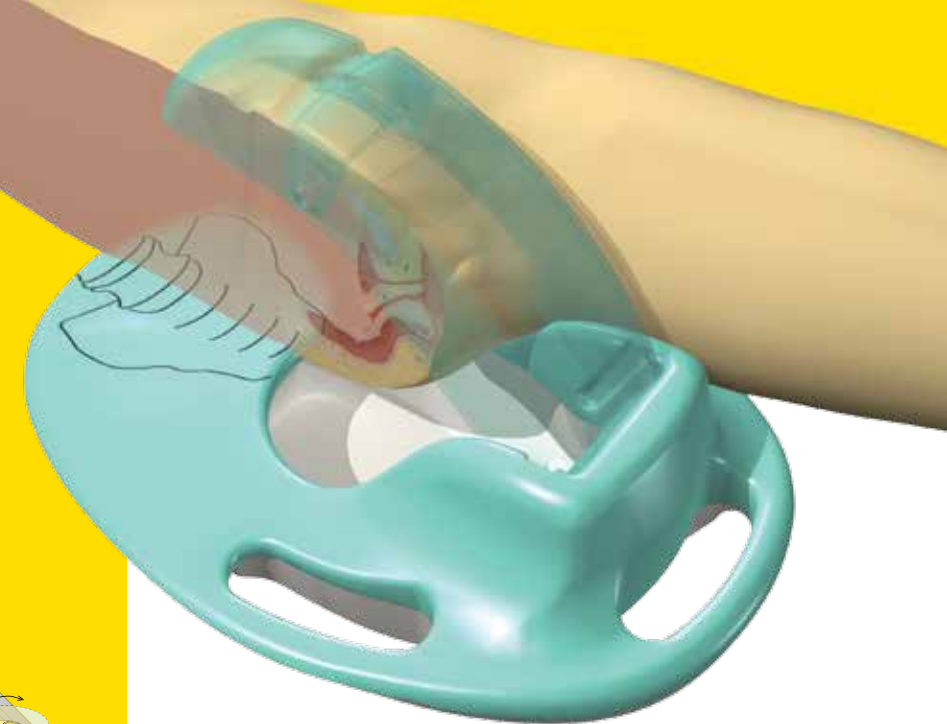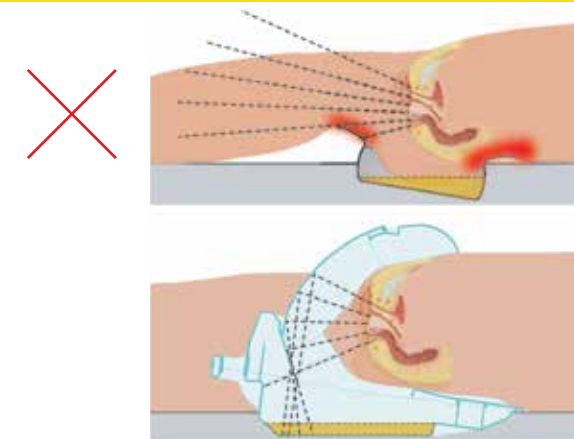

Less stress for nurse and patient

Using Multiclean Bedpan a longer period, patients appreciate the warm and ergonomically formed plastic. Multiclean Bedpan will be ordered by the patient in time, because patients are no longer afraid and recognise the high lying comfort. This prevents stress for patient and nurse.

Cleaning Multiclean bedpan :

- Washing with washing machine
- Manual washing
- Can disinfected with ordinary autoclaving methods or skin and plastic secure cleaners and disinfectants.

